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1.0 Foreword:

Dear customer,

We are pleased that you have chosen a quality product from the range of rehastage GmbH. A good relationship with our customers is important to rehastage. Customer proximity means fast service, as little bureaucracy as possible and close cooperation with our customers. Seek advice from your treating physician or therapist to ensure that this product is suitable for you and meets your needs.

- ⚠ **For your personal safety, please read these instructions carefully before using the product. If there are any problems understanding the information, warnings and instructions listed here, please inquire with your specialist dealer to avoid injury or property damage due to incorrect operation.**

Do not throw away these instructions for use even after the product has been assembled, so that you can use them later if necessary. If you need spare parts or accessories or simply have a question about your product, we are here for you.

We want you to be satisfied with our products and service. We at rehastage are therefore constantly working on the further development of our products. For this reason, changes in shape, technology and features may occur in our product range. Therefore, no claims can be derived from the information and illustrations in this instruction manual.

2.0 Intended use:

The sofa handle Deluxe "Couch Tray" is designed to support controlled getting up and sitting down as well as a standing aid. The existing tray offers the additional possibility of using the sofa handle Deluxe "Couch Tray" as a worktable, e.g. when using a notebook, as a shelf or for taking meals.

The aim is to enable the user to lead an independent or more independent life in everyday life.

- Indication:

The sofa handle Deluxe "Couch Tray" serves users.

- with functional disorder of the lower extremities due to limited or completely abolished mobility of the joints,
- in case of restriction of muscular strength of the lower extremities,
- In the case of damage to the lumbar spine with impairment of the joints, mobility with sufficient functionality of the upper extremity and sufficient trunk stability
- with generally reduced or impaired mobility without age restrictions.

- Contraindications:

- ⚠ Additional safety measures should be taken for people who belong to certain risk groups. This includes people with a mental impairment (including those under the influence of medication), very tall people or people who are severely overweight, as well as any condition that affects a person's ability to use the Deluxe Couch Tray Sofa Handle without assistance.

- ⚠ If you have a medical condition or injury or are taking medications that affect your sense of balance or your ability to stand or sit, please consult your physician before using this product. Improper use of the product may result in serious injury or damage.

- ⚠ This product is only intended as a balance aid and support when sitting and standing, it is not intended to support the full weight of a person.

3.0 Other precautions and warnings:

Please read the following instructions carefully before using the product.

- ⚠ For the sofa handle Deluxe "Couch Tray", it is important that the sofa or armchair is heavy enough to hold the product. The sofa handle Deluxe "Couch Tray" is not suitable for light sofas or armchairs and should therefore not be used with them.

- ⚠ Regularly check the screw connection and locks as well as the positioning of the sofa handle Deluxe "Couch Tray" under the seat.

- ⚠ The product is not sports equipment! It is not suitable for performing movement exercises or other sporting activities of any kind on it.

- ⚠ The product is not a toy! Make sure that children do not use this product as a playground device or hang on to it.

- ⚠ The instructions accompanying the product must be followed carefully to ensure that the product is properly installed and constructed. Failure to follow the instructions may result in serious injury to the user or damage to the product.

- **DISCLAIMER:** Neither Stander Inc. nor the manufacturer/distributor shall be liable for any damages or injuries that may result or have resulted from the improper use or assembly of the product.

- In case of installation problems, please contact the selling office (specialist dealer) or directly: Rehastage GmbH, Artlandstraße 73, 49610 Quakenbrück / Germany, qm@rehastage.de

4.0 Scope of delivery:

Part	Naming	Pcs.	Part	Naming	Pcs.
A	Handlebar with bracket and screws J	1	E	Tray with bracket and screw K	1
B	Standpipe	1	F	Allen wrench size 6	1
C	Front base plate	1	G	Screw	1
D	Bottom plate at the rear	1	H	Allen key size 4	1

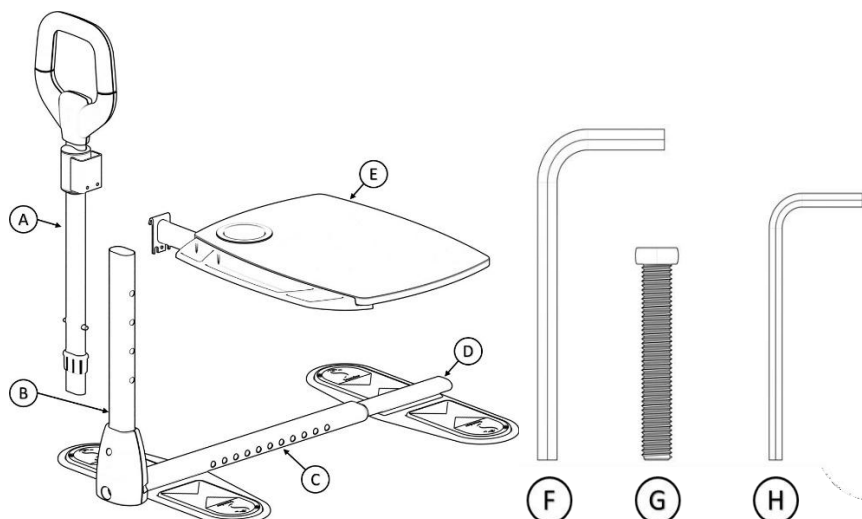
Figure 1

5.0 Assembly and assembly instructions:

5.1 Remove the product from the box and identify all the parts as shown in Figure 1.

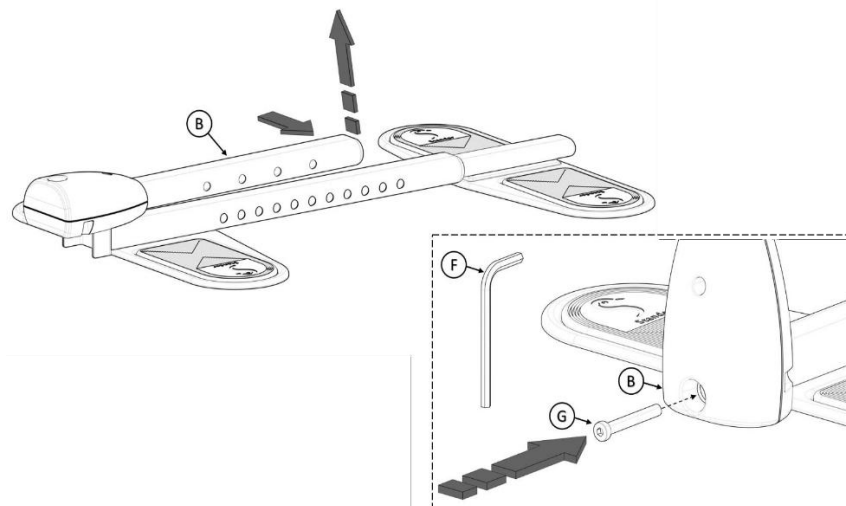
Figure 2

5.2 Fold the standpipe **B** to a vertical position. Insert the screw **G** through the hole in the front of the plastic cover on the foot and screw the screw into the thread in the bottom plate at the front. Screw the screw tightly with the Allen wrench size 6 (**F**). See Figure 2

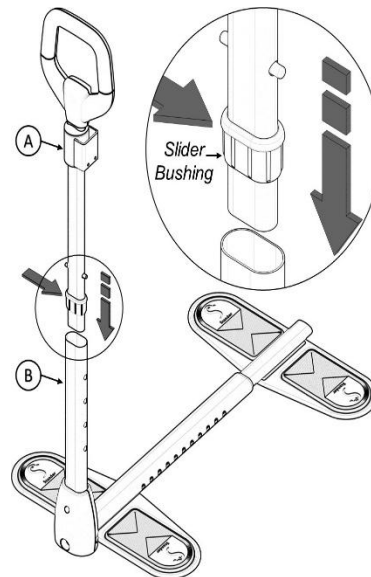
Figure 3

- 5.3** Take the handlebar **A** and insert it into the standpipe **B**. Push the slider bushing into the stanchion **B** as far as it will go. Press the spring pins together and push the handlebar **A** down into the stanchion **B** as far as possible. See Figure 3

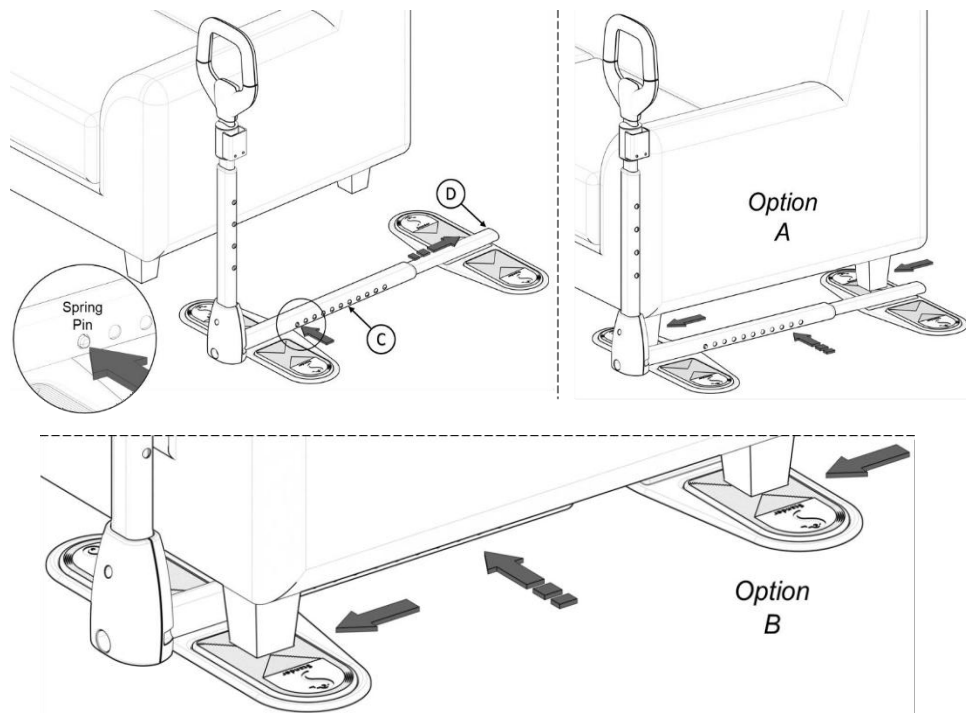
Figure 4

- 5.4** Place the mounted "Couch Tray" next to the seating furniture intended for use (chair, armchair, couch, etc.). Pull the front and rear floor plates (**C** and **D**) apart so that the rubber pads of the floor plates are parallel to the furniture feet.
- Lift the seat and slide the base plate under the furniture feet on the inside or outside. See Figure 4
- The legs of the seating furniture should be in the middle of the rubber pads of the floor plates **C** and **D**. The base of the stanchion **B** should be placed as close as possible to the seat at the front.
 - Make sure that the spring pins are fully engaged in the corresponding hole.

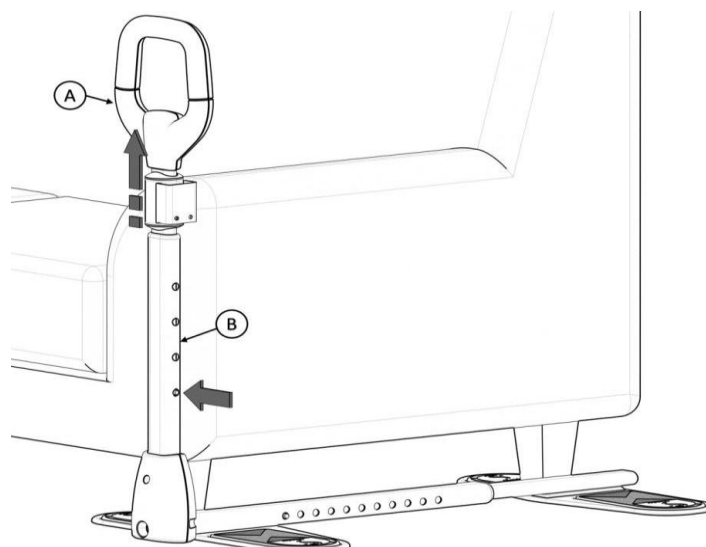
Figure 5

5.5 Now adjust the handlebar **A** to the height you want.

Make sure that the spring pins are fully engaged in the corresponding hole.

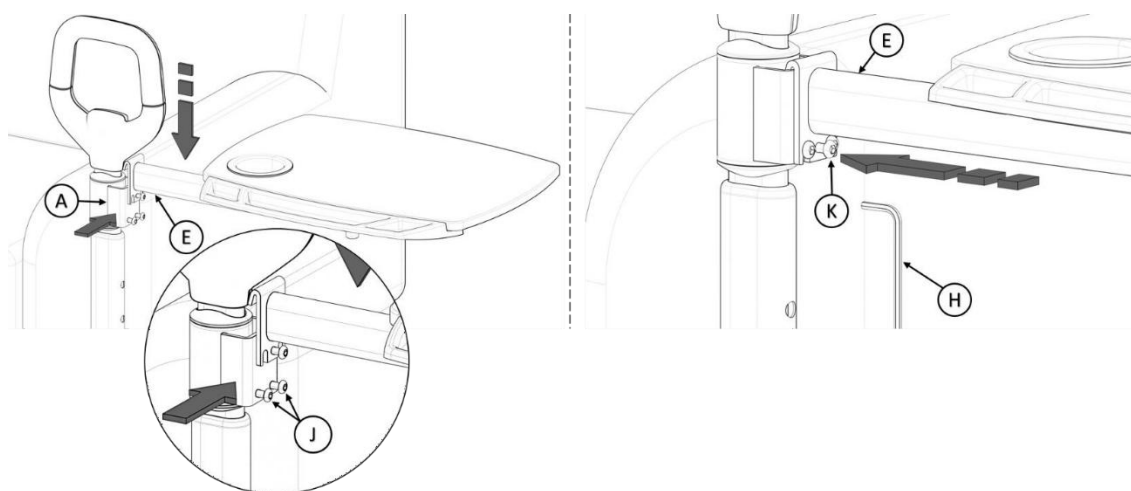
Figure 6

5.6 Loosen the screws **J** on the holder of the handlebar **A**. Hook in the bracket of tray **E**. By screwing the screw **K** in or out, place the tray **E** horizontally. Then tighten the screws **J** again with the Allen wrench size 4 (**H**). See Figure 6

6.0 Cleaning instructions

Clean the Deluxe "Couch Tray" sofa handle regularly with a damp sponge, mild soap and a standard household disinfectant. Dry it with a dry soft cloth. Do not use abrasive cleaning agents such as scouring milk or powder, as this can lead to destruction or damage to the surfaces and threads.

7.0 Specifications:

Handle height:	86 – 102 cm (34" – 40")
Tray height:	66 cm – 81 cm (26" – 32")
Tablet size:	40.6 x 45.7 cm (16" x 18")
Load capacity tray:	13.6 kg (30 lbs)
Total load capacity:	113.4 kg (250 lbs)
Dead load:	11.3 kg (25 lbs)

8.0 Disposal / recycling of materials

The following section describes the materials used for the aid, with details on how to dispose of or recycle the tool and how to package it. Special regulations regarding disposal or recycling may also apply locally, these must be considered during disposal.

- Galvanized steel: Base body and add-on parts partly powder-coated, fastening screws
- Plastic: Anti-slip rubbers, bags for mounting material, packaging material
- Cardboard: Packaging

Disposal should be carried out by a waste management company or a public disposal point.

9.0 Warranty

In addition to the statutory warranty, rehastage offers a warranty on this product against manufacturing or material defects of 2 years from the date of purchase. The warranty is only valid against the first user of the item. Keep the invoice that is your proof of purchase. The warranty does not cover defects caused by wear and tear, improper handling or commercial use of the device. The warranty is void if the item is repaired or altered by anyone other than rehastage itself or a dealer authorized by rehastage. The warranty obligation expires if repairs or modifications or replacements of individual parts that are not original rehastage spare parts are carried out by a party other than the seller or the supplying company. During the warranty period, rehastage GmbH is responsible for the decision to repair the device free of charge or to replace it free of charge.

10.0 Explanation of symbols used markings and information to be provided

	Manufacturer, name and address of the manufacturer.
	Importer, name and address of the importer.
	Follow the instructions for use! (if required)
	Attention! Information about warnings, safety measures, or precautions
	Hint! Tips and advice, on use and function.

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